

## **Phlebotomist's Guide for Baseline Mr. OS Blood Collection**

We have created this guide as lab reference material. It is only a supplement to the operations manual chapter. The operations manual chapter has more information. This guide acts as only a quick reference.

We are expecting that the staff collect all of the necessary tubes for each participant. However, different situations will occur and we have described every possible scenario that could happen with the two draws from the baseline Mr. OS specimen collection protocol.

Draw tube #1 (from 1 <sup>st</sup> draw) Draw tube #2 (from 1 <sup>st</sup> draw) Draw tube #3 (from 1 <sup>st</sup> draw) Draw tube #4 (from 1 <sup>st</sup> draw) Draw tube #5 (from 1 <sup>st</sup> draw) Draw tube #6 (from 2 <sup>nd</sup> draw)	This is the recommended blood collection protocol. All deviations from this protocol are described on the following page.
--	---

If on the first draw you collect.....  
should.....

on the second draw you

The ideal scenario: Draw tubes #1, #2, #3, #4, #5	Collect draw tube #6.
Draw tube #1 only	attempt to collect draw tube #6 first and then collect draw tubes 2,4,5. <b><u>Do not collect draw tube #3.</u></b>
Draw tubes #1, #2 only	attempt to collect draw tube #6 and then collect draw tubes 4,5. <b><u>Do not collect draw tube #3.</u></b>
Draw tubes #1, #2, #3 only	Collect draw tube #6 first, then collect draw tubes #4, #5.
Draw tubes #1, #2, #3, #4	Collect draw tube #6 first, then collect draw tube #5.